# REF 5090250 TECHNOVIEW® Edoxaban Calibrator Set 5x1mL

3015601 Rev. 002 [01] 03/2018

## TECHNOVIEW® Edoxaban Calibrator Set 5x1mL- English

### INTENDED USE

TECHNOVIEW® Edoxaban Calibrator Set is a set of 5 calibration plasmas to be used for calibration of edoxaban measurements, optimized using TECHNOCHROM® anti-Xa assay on Ceveron® alpha.

### SUMMARY

Edoxaban (Savaysa®, Lixiana®) is an oral anticoagulant drug which acts as a direct factor Xa inhibitor.

### REAGENTS

The TECHNOVIEW® Edoxaban Calibrator Set 5x1mL contains:

mL	Reagent	Description
1 x 1	CAL 1	Calibrator 1, human plasma, lyophilized, no edoxaban
1 x 1	CAL 2	Calibrator 2, human plasma, lyophilized, ~50 ng/mL edoxaban
1 x 1	CAL 3	Calibrator 3, human plasma, lyophilized, ~140 ng/mL edoxaban
1 x 1	CAL 4	Calibrator 4, human plasma, lyophilized, ~300 ng/mL edoxaban
1 x 1	CAL 5	Calibrator 5, human plasma, lyophilized, ~500 ng/mL edoxaban

Please consult the batch table provided with the calibrator set for the exact concentrations in ng/mL of each calibrator.

### Material required (not supplied with the kit)

- Pipettes
- Distilled water

REF 5340250 TECHNOCHROM® anti-Xa Kit 80.T

REF 5090251 TECHNOVIEW® Edoxaban CONT Low 5x1mL

REF 5090252 TECHNOVIEW® Edoxaban CONT Medium 5x1mL

REF 5090253 TECHNOVIEW® Edoxaban CONT High 5x1mL

### Warning and precautions

IVD - for in vitro plasma use.

Every single donor plasma and every lot of the calibrators included is tested and found negative for HbSag, HIV 1/2 antibodies and HCV antibodies. However, general precautions should be taken by handling all human source materials as potentially infectious.

All blood and plasma samples and products have to be regarded as potentially infectious and handled with appropriate care and in compliance with the biosafety regulations in force. Dispose the same way as hospital waste.

A Material Safety Data Sheet for this product is available from [www.technoclone.com](http://www.technoclone.com)

### Stability and storage

The expiry date printed on the labels is only applicable to storage of the unopened containers at 2...8°C. Stability after reconstitution:

Reagent	18...25°C
CAL 1	4 hours
CAL 2	4 hours
CAL 3	4 hours
CAL 4	4 hours
CAL 5	4 hours

### TEST PROCEDURE

#### Preparation of reagents

Vials with lyophilized calibrators and distilled water used for reconstitution need to reach room temperature (18...25°C) before reconstitution. Reconstitute each vial (calibrator 1-5) with 1 mL of distilled water. Allow the reconstituted material to stand at room temperature for 30 minutes followed by swirling of the vial before use. Avoid foaming.

#### Performance of the test

The TECHNOVIEW® Edoxaban Calibrator Set is always used in combination with the TECHNOVIEW® Edoxaban Control Low, Medium and High and the TECHNOCHROM® anti-Xa kit.

The edoxaban concentration of calibrator 1-5 may vary from one lot to another but is clearly indicated in the lot specific batch table provided in the calibration set.

To cover the whole assay range, two applications are established for this product. For Standard application, a calibration curve is established using calibrator 1, calibrator 2 and calibrator 3. All samples are assayed with this application. Results below the limit of quantification (instrument specific) should be declared as <LoQ. Samples found with a concentration above calibrator 3 in standard application should be re-tested using application high. For the application high, calibrator 1, calibrator 3, calibrator 4 and calibrator 5 are used to establish a calibration curve. Technoclone provides instrument specific application sheets which contain analyser/assay specific handling and performance information.

### STANDARDISATION

The TECHNOVIEW® Edoxaban Calibrator Set is directly traceable to high performance liquid chromatography with mass spectrometry detection (HPLC-MS).

### LITERATURE

Please contact Technoclone [www.technoclone.com](http://www.technoclone.com) or your local distributor.

Savaysa® and Lixiana® are registered trademarks of Daiichi Sankyo.

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Technoclone Herstellung von Diagnostika und Arzneimitteln GmbH, Brunner Strasse 67, 1230 Vienna, Austria