


NEW !

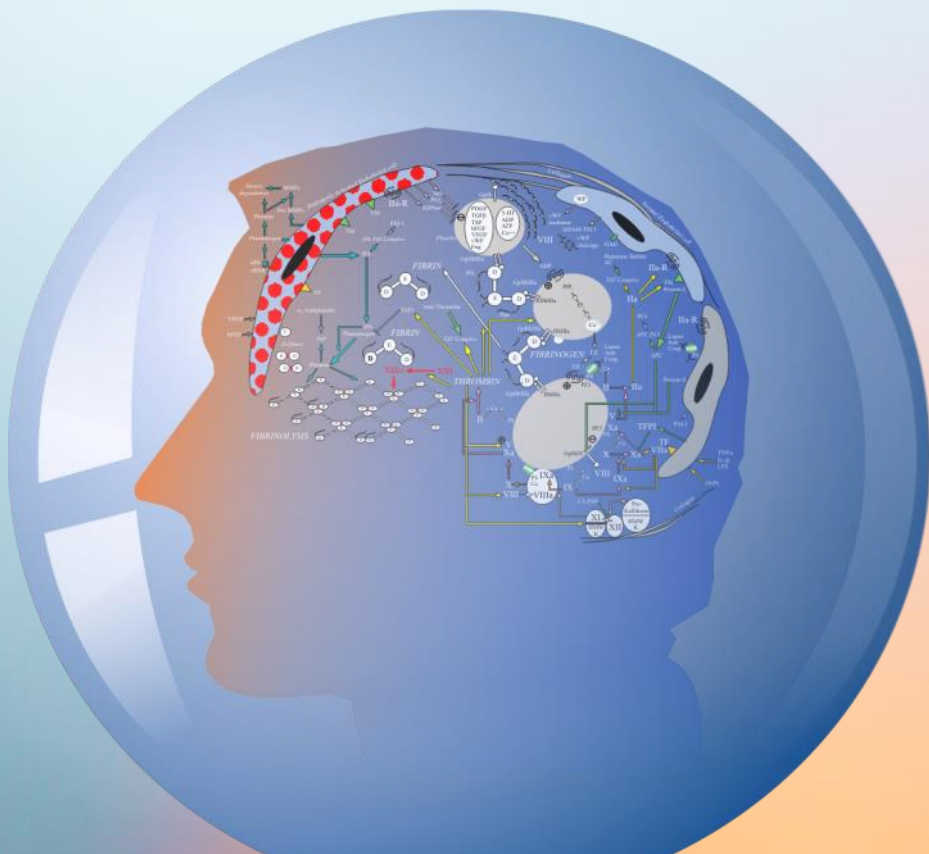

technoclone
TIC

EDOXABAN Measurement

Reagents for accurate measurement of the new oral direct Xa Inhibitor Edoxaban

A clear view into the future!





TECHNOVIEW® EDOXABAN CALIBRATORS

The **TECHNOVIEW® Edoxaban Calibrator Set** contains 5 calibrators with different concentrations of Edoxaban (Lixiana®, Savaysa®) covering the measurement range ~0-500 ng/mL.


TECHNOVIEW® EDOXABAN CONTROLS

The **TECHNOVIEW® Edoxaban Controls** are normal plasma samples with low (~30 ng/mL), medium (~125 ng/mL) and high (~400 ng/mL) concentration of Edoxaban (Lixiana®, Savaysa®).

PERFORMANCE OF TECHNOCHROM® ANTI-Xa WITH EDOXABAN

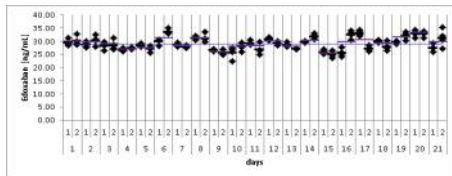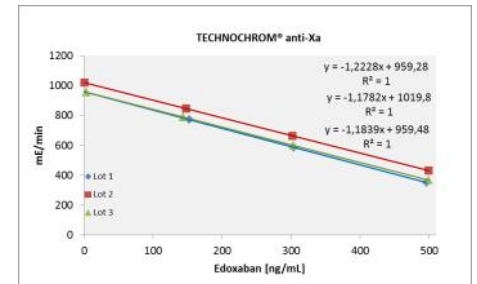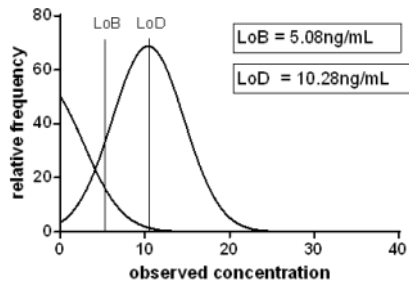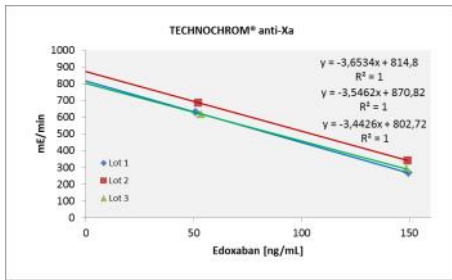


LoQ is determined at 15 ng/mL

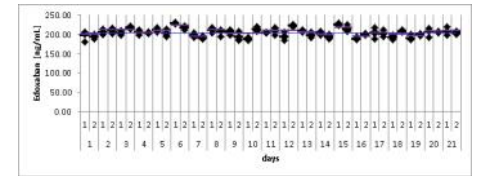

CORRELATION WITH LC/MS RESULTS

Excellent correlation of spiked samples as well as patient samples showing a Passing & Bablock regression with a slope of 1.0047, an Intercept < 5.5 ng/mL and $r > 0.95$.

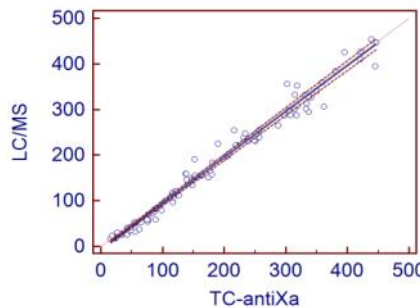

Regression Equation	
$y = -5.5258 + 1.0047 x$	
Intercept A	-5.5258
95% CI	-8.0269 to -3.1221
Slope B	1.0047
95% CI	0.9827 to 1.0252
Cusum test for linearity	No significant deviation from linearity (P>0.10)

Product	Description	REF	Package
TECHNOCHROM® anti-Xa	1 x 20 ml R1 anti-Xa Buffer		
	1 x 4 ml R2 bovine Faktor Xa	5340250	~ 80 Teste
	1 x 4 ml R3 Substrate		
TECHNOVIEW® Edoxaban	Calibrator Set with 5 Calibrators (~ 0-500 ng/ml)	5090250	5 x 1 ml
	Control Low (~ 30 ng/ml)	5090251	5 x 1 ml
	Control Medium (~ 125 ng/ml)	5090252	5 x 1 ml
	Control High (~ 400 ng/ml)	5090253	5 x 1 ml

I'm not afraid of DOACs!

Technoclone Herstellung von Diagnostika und Arzneimitteln GmbH
 Brunner Str. 67 | 1230 Vienna | Austria
 Phone: +43 1 86373-0 | Fax: +43 1 86373-44
 sales@technoclone.com | www.technoclone.com

